CERRAHİ ALETLER SİSTEMİNİN KULLANIM TALİMATI

SURGICAL INSTRUMENTS SYSTEM INSTRUCTIONS FOR USE (IFU)

(EN-TR)

CE 2803

EN

HIPKNEE Knee and Hip Prosthesis Application Sets Cleaning Sterilization and User Manual

General Description:

Hipknee Surgical Instruments are a set of general and unique instruments specially designed for orthopedic surgery. Such tools make it easier for orthopedic surgeons to accomplish, enable or accelerate certain tasks related to surgical techniques and implant systems. Hipknee Surgical Instruments are reusable or disposable devices. Generally, disposable devices perform puncturing, cutting, shearing and puncturing operations occurring in musculoskeletal structures. Reusable tools perform other functions that occur outside of musculoskeletal structures: fixing, bending, twisting, striking, insertion/removal/retraction, grasping, repositioning, alignment/targeting, try/templating, stretching, torqueing, cutting, clamping, lifting and measuring. Hipknee Surgical Instruments and implants are presented to the user in a system-specific, non-sterile rigid instrument tray that accommodates specific Hipknee devices to be processed for examination, cleaning, sterilization, transportation and storage.

Key Design Features: Hipknee Surgical Instruments are manufactured from medical grade stainless steel, aluminum, and common medical device polymers. Tools can be hand-held, manually operated, or connected to electrical equipment where applicable. Hipknee Surgical Instruments should only be used in conjunction with Hipknee devices suitable for surgical technique (e.g. interactive instruments and implants).

Indications, Contraindications, Adverse Effects:

Hipknee Surgical Instruments should only be used for their intended purpose. Surgical technique provides the method of administration for basic instrumentation. The decision to use a tool should consider factors such as surgical practice and physical and mechanical properties. If a more comprehensive understanding of device mechanics or method of use is needed, medical personnel are encouraged to contact Hipknee representative(s). In case of premature failure of a device for which a causal relationship with its geometry, surface quality or mechanical stability is suspected, please Deliver in a cleaned, disinfected condition according to Hipknee's cleaning and sterilization instructions.

Instructions for Use and Reprocessing

Definition Device for sale, distribution and use by or on the order of a physician. Reusable, non-sterile device.

Cleaning Before Use

Warning: What is recommended here, as well as your facility's policies and procedures, or your sterilizer manufacturer's designated location, should be brought to the attention of responsible persons.

cleaning and sterilizing your instruments, your orientation facility. All instruments should be checked to ensure proper function and condition before use. Do not use tools if they do not work
Carefully remove all protective covers and cases. Before using the instrument, the instruments must be cleaned, decontaminated, sterilized and checked before use in surgery.

Reprocessing Limitations

Repeated rework has minimal impact on instruments. The service life is normally subject to wear and damage caused by use.

Caution: Risk of Damage - Surgical instruments are precision instruments. Careful use, correct operation. Improper external use (bending, hitting, dropping, etc.) may result in product failure.

Check Function Before Use

The general functioning and preparation of tools and accessories should be checked before use.

Please check before use.

Operation

Surgical procedures should only be performed by individuals with sufficient training and experience.

Additionally, consult medical literature regarding technique, complications, and hazards.

All instructions regarding safety features and surgical techniques should be read carefully before using the product. The device should only be operated by trained personnel.

Possible Adverse Effects:

The potential for adverse reactions exists with any surgical procedure. Possible adverse effects specific to surgical instruments are listed below. These do not include all adverse effects that may occur with surgical procedures. Compromise of mechanical, physical or chemical properties resulting in: a. Regarding the patient i. Bone damage or necrosis ii. Inflammatory, foreign body or allergic reaction iii. Infection b. Device related i. Device loosening, splitting, bending, peeling, cracking or breaking ii. Fracture leading to device fracture fragments within the wound iii. Loss or deficiency of function or attachment mechanisms the adverse effects listed here are not specific to Hipknee Surgical Instruments

Please follow general operating room technique.

Cleaning / Sterilization

point of use

Excess dirt should be removed immediately after use with a disposable cloth, wipe or gauze.

Collection and Transport

It is recommended that devices be reprocessed as soon as reasonably possible after use.

Cleaning and Sterilization Preparation

Simple assemblies need to be disassembled to ensure more complete cleaning and sterilization.

Disassembly should not require any mechanical tools (e.g. screwdriver, pliers, etc.).

Follow the instructions below before sterilization.

Manual (Required if not using automatic washer/disinfectant method)

- All instruments should be cleaned in a fully opened and disassembled configuration.

- Neutral according to supplier's instructions pH Prepare enzymatic detergent. Highly alkaline detergents are recommended for aluminum appliances.

- Immerse the device completely in the detergent prepared according to the labeling instructions.

- Activate all moving parts during the soaking period to ensure that the detergent penetrates completely into hard-to-reach areas.

- Scrub the device using a soft-bristled brush until all visible dirt is removed, paying attention to moving parts, crevices, lumens and other hard-to-reach areas

ultrasonic flush ports for flushing.

- Additionally, neutral for lumen devices pH Prepare enzymatic detergent (according to vendor) and ultrasonically wash the instruments for at least 10 minutes. Note: Changes when the enzyme solution becomes extremely contaminated (bloody and/or cloudy).

Rinse all surfaces, crevices, and lumens in RO/DI running water for at least three (3) minutes to remove any residual detergent or debris.

- Dry the device with a clean, soft cloth.

- Visually inspect each device for cleanliness. If visible dirt remains, repeat the cleaning process.

Cleaning: Automatic

- All instruments must be completely open and cleaned.

- All instruments must be cleaned in a fully open and disassembled configuration.

all devices in a washer/ disinfectant using the equipment and detergent recommended by the equipment.

- Visually inspect each device for cleanliness. If visible dirt remains, repeat the cleaning process.

distilled or deionized water to remove any remaining solution.

Inspection and Function Test

- Check if the hinges move smoothly. There should be no scratches on the locking mechanisms.

- Tools with broken, cracked, chipped or worn parts or blackened surfaces should not be used. They must be repaired or replaced immediately.

Packaging

Devices may be wrapped in sterilization wrap according to the manufacturer's instructions.

Sterilization: Autoclave

Our devices are reusable and meet standards for sterilization. Sterilization of instruments can be done by Autoclave or Ethylene Oxide. Time and temperature

CERRAHİ ALETLER SİSTEMİNİN KULLANIM TALİMATI
SURGICAL INSTRUMENTS SYSTEM INSTRUCTIONS FOR USE (IFU)
(EN-TR)

CE 2803

parameters vary depending on the type of sterilizer, cycle design and packaging material required for sterilization.

Instrument sets for sterilization can be accepted by every healthcare institution. sterilize devices at temperatures above 141°C (285°F).

- All ring-handled instruments should be autoclaved in the fully open position to prevent the case from cracking.
- All instruments must be sterilized in a fully open and disassembled (i.e. disassembled) configuration.
- All devices should be positioned to ensure that all surfaces come into contact with steam.
- All tools with concave surfaces should be placed so that the surfaces drain water.
- Always verify parameters with the sterilizer manufacturer's written instructions.
- It is not recommended to use "flash" sterilization as it may shorten the life of the devices.

Cleaning and Sterilization

Parameters for Wrapped Instruments in Steam Sterilization

	Heat	exposure	Drying
Gravitational Displacement	132°C (270°F)	15 minutes	15 minutes
Front Vacuum / Dynamic	132°C (270°F)	4 minutes	15 minutes
Air Purification	135°C (275°F)	3 mins	15 minutes

Storage

- After sterilization, instruments should remain in the sterilization packaging and in a clean, dry cabinet or storage bin.
- Care should be taken to prevent damage to the containers.
- Make sure the instrument container is sealed in packaging suitable for sterilization. Sterilization should be carried out in accordance with local guidelines for hospital hygiene.

Care

Caution: Apply lubricant only to fasteners (locking mechanism) and moving parts.

Repair

ensure that all repairs are completed in accordance with the manufacturer's specifications, the device should only be repaired by Endoven or an authorized service facility.

Regardless of age, if any device needs service, send it to your authorized service center.

Devices must be cleaned and sterilized before shipping.

MEANING OF SYMBOLS:

	Consult the user manual.
	Lot numbers
	Product code number (Catalog number)
	Non-sterile
	Manufacturer name and address
	Production date
	European union symbol of conformity
	Medical device
	Indicates a carrier that contains unique device identifier information
	Caution
	Keep dry

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CERRAHİ ALETLER SİSTEMİNİN KULLANIM TALİMATI

SURGICAL INSTRUMENTS SYSTEM INSTRUCTIONS FOR USE (IFU) (EN-TR)

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